



Your blood glucose result is lower than 1.1 mmol/L (20 mg/dL) or there may be a problem with the electrode.

**Repeat test with a new electrode.*



Your blood glucose result is higher than 27.8 mmol/L (500 mg/dL) or there may be a problem with the electrode.

**Repeat test with a new electrode.*



Your blood glucose result is 16.7 mmol/L (300 mg/dL) or higher. If you check your ketones as part of your diabetes management program, it is recommended that you check your blood β -Ketone.



Temperature is too hot or too cold for the system to work properly.
Move monitor and electrodes to a location with an appropriate temperature.



Monitor error.
Turn monitor off; repeat prior steps.



There may be a problem with the electrode.
Repeat test with a new electrode.



Glucose result may be too high to be read by the system, or there may be a problem with the glucose or β -Ketone electrode.

**Repeat test with a new electrode.*



Blood applied to electrode too soon.
Repeat test with a new electrode.

**If the message appears again, contact your healthcare professional immediately.*

FOLD

FOLD

FOLD



Calibration/electrode error.

Repeat calibration steps.



Monitor error.

Repeat test with a new electrode.



Electrode error.

Electrode is damaged, used, or the monitor does not recognize it.

Repeat test with a new electrode.



Monitor error.

Repeat test with a new electrode.

These instructions are intended as a quick reference guide only. For complete operating instructions and other important product information, refer to the Optium™ Xceed™ User's Guide and the MediSense® Optium™ Blood Glucose and MediSense® Optium™ Ketone Blood β-Ketone electrode instructions for use.

Customer Support

Australia: 1800 801 478

New Zealand: 0800 106 100

Reorder No.: 70315-01

Abbott Diagnostics Division, MediSense Products,
666 Doncaster Road, Doncaster 3108, Victoria, Australia

MediSense, Optium and Xceed are trademarks of
Abbott Laboratories. ©2004 Abbott Laboratories.

P/N 133-225 Rev. 0 5/04



* 1 3 3 - 2 2 5 R E V . 0 5 / 0 4 *

MediSense®

Optium™ Xceed

Diabetes Monitoring System

Display Symbols and Messages



Slug Does Not Print On Final. For Labelling Production Use Only

Part Number	133-225	MAC	
Rev #	0	PROD	
Creative #		TRAF	
Date Open	13 Apr 04	A/S	
LCN# C	Target Prep	A/D	
Last Edited	11 May 04	ORIG	